


ColActive® PLUS Ag

Collagen Matrix Dressing with Silver

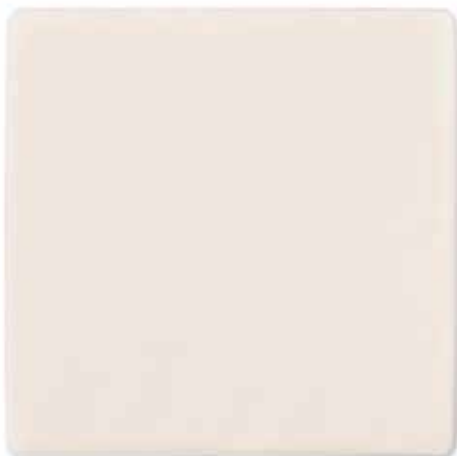

✓ FACILITATES NATURAL WOUND HEALING

EDTA and collagen in ColActive® Plus Ag target and deactivate elevated MMP Activity*.

✓ MOIST WOUND ENVIRONMENT

Alginate and CMC in ColActive® Plus Ag allows the dressing to absorb up to 20 times its weight in wound drainage, while still maintaining an optimal moisture balance for wound closure*.

✓ ANTIMICROBIAL SILVER

ColActive® Plus Ag contains ionic silver for broad spectrum antimicrobial activity.

✓ CLINICAL TRACK RECORD

Over ten years of clinical success in treating chronic stalled wounds.

HOW DOES IT WORK?

ColActive® Plus Ag Collagen Matrix Dressing with Silver is an advanced wound care dressing made of collagen, sodium alginate, carboxymethylcellulose (CMC), ethylenediaminetetraacetic acid (EDTA) and silver chloride. The topically applied wound dressing will transform into a soft gel sheet when in contact with wound exudates. The dressing maintains a moist wound environment at the wound surface that aids in the formation of granulation tissue and epithelialization. The EDTA in the dressing removes zinc to inhibit the activity of matrix metalloproteinases (MMP), thus creating a suitable environment for wound healing. The silver chloride in the dressing provides effective broad spectrum antimicrobial activity. The dressings may be trimmed and layered for the management of deep wounds.

WHEN TO USE:

ColActive® Plus Ag is indicated for management of full and partial thickness wounds including: pressure ulcers, diabetic ulcers, ulcers caused by mixed vascular aetiologies, venous ulcers, donor and graft sites, abrasions, traumatic wounds healing by secondary intention, dehisced surgical wounds, first and secondary degree burns.

WHEN NOT TO USE:

ColActive® Plus Ag dressing should not be used on third degree burns or on patients with a known allergy or sensitivity to animal derived collagen, silver, or any other ingredient.

*Covalon Technologies Ltd., Data on file.



ColActive® PLUS Ag
Collagen Matrix Dressings with Silver

1-877-711-6055 |


covalon
TECHNOLOGIES LTD.

ColActive® Plus Ag may not be cleared in all markets and product indication claim(s) may vary between markets. ColActive is a registered trademark of Covalon Technologies Ltd. © 2018 Covalon Technologies Ltd. Mississauga, ON, Canada L4W 5S7.

ColActive[®] PLUS Ag

HOW TO APPLY:

1. Debride and irrigate the wound bed with an appropriate wound cleanser or distilled water before application.
2. Cut the dressing to fit the exact wound size. For heavily exudating wounds, apply the dressing directly to the wound bed. For dry wounds with very little exudate, moisten the wound bed with distilled water to begin the gelling process.
3. Cover the ColActive[®] Plus Ag with an appropriate secondary dressing.
4. Change dressings daily or as good nursing practice dictates.

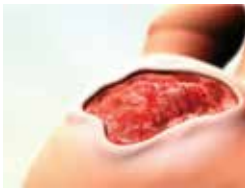

1.



2.



3.



4.

CARE AND REMOVAL:

1. Remove the secondary dressing with care. Gently remove ColActive[®] Plus Ag ensuring that the dressing removal does not damage any newly formed tissues.
2. Use an appropriate wound cleanser or distilled water to cleanse the wound site prior to applying a new ColActive[®] Plus Ag dressing as per application instructions.

AVAILABILITY:

Reference Number	HCPCS	Size (cm)	Dressings per Carton / Cartons per Case
TWBC1033 5551220	A6021 —	5cm x 5cm	10/10
TWBC1034 5551440	A6022 —	10cm x 10cm	10/10
TWBC1032 5551770	A6023 —	18cm x 18cm	10/10

NOTE:

- Please use TWBC reference numbers for U.S. CMS reimbursement.
- HCPCS codes contained in this document are provided for informational purposes only and represents no statement, promise, or guarantee by Covalon Technologies Ltd. concerning levels of reimbursement, payment, or charge.
- As with any medical device, always refer to the full instructions for use prior to application.



ColActive[®] PLUS Ag
Collagen Matrix Dressings with Silver

1-877-711-6055 |

covalon
TECHNOLOGIES LTD.