

CovaWound™ Foam

Non-Adherent Foam Dressing



- ✓ **BREATHABLE POLYURETHANE FILM BACKING**
Allows moisture vapour transmission away from the dressing. Designed to protect against eschar formation and is also waterproof, protecting against external contamination.
- ✓ **HIGHLY ABSORBENT POLYURETHANE FOAM**
Designed to absorb and retain exudate, while still maintaining a moist wound environment.

WHAT IS IT?

CovaWound™ Foam Non-Adherent Foam Dressing is a non-adherent foam dressing with a two layer construction. CovaWound™ Foam Non-Adherent Foam Dressing is soft and conformable providing an optimal moist healing environment and excellent patient comfort.

WHEN DO I USE IT?

CovaWound™ Foam does not adhere to the wound and, therefore, is recommended for wounds surrounded by sensitive skin including venous ulcers, the management of moderate to heavily exuding wounds such as pressure, leg, diabetic and foot ulcers, superficial and partial-thickness burns, donor sites, trauma wounds and post-operative surgical wounds.



CovaWound™
Advanced Wound Care

1-877-711-6055 | covawound@covalon.com

covalon
TECHNOLOGIES LTD.

HOW TO APPLY:

1. Clean the wound area with normal saline or distilled water.
2. Gently dry the surrounding skin and ensure that it is free of any residue.
3. Choose the size and shape of the dressing which is most suitable, the dressing should extend 2 cm beyond the wound margin.
4. Remove the dressing from the packaging.
5. Remove the release liner from the dressing and apply the white side of the non-adherent foam dressing to the wound.
6. A retention method such as taping onto intact skin or bandaging in place with an appropriate bandage or a film dressing is acceptable.

FREQUENCY OF CHANGE:

CovaWound™ Foam dressing may be left in place for up to 7 days depending on the amount of exudate and the condition of the wound. CovaWound™ Foam dressing should be changed when visible signs of exudate approach the edge of the dressing. The accumulative use of CovaWound™ Foam dressing should not exceed 30 days.

NOTE:

- As with any medical device, always refer to the full instructions for use prior to application.
- HCPCS codes contained in this document are provided for informational purposes only and represents no statement, promise, or guarantee by Covalon Technologies Ltd. concerning levels of reimbursement, payment, or charge.

AVAILABILITY:

COVAWOUND™ FOAM

Non-Adherent Foam Dressing

Exudate Level Key: Heavy



Reference Number	Size (cm)	Dressings per Carton/ Cartons per Case	HCPCS
2000505	5cm x 5cm	10/20	A6209
2000510	5cm x 10cm	10/20	A6209
2000520	5cm x 20cm	10/20	A6209
2001010	10cm x 10cm	10/20	A6209
2001020	10cm x 20cm	10/20	
2001515	15cm x 15cm	10/10	A6210
2002020	20cm x 20cm	10/10	A6211
2001013 (Heel)	10cm x 13cm	5/20	



Suggested Wound Tissue Type:

Necrotic (After Debridement)	✓
Sloughy	✓
Granulating	✓
Epithelialising	✓
Infected	—

Availability may vary by jurisdiction.



CovaWound™
Advanced Wound Care

1-877-711-6055 | covawound@covalon.com

covalon
TECHNOLOGIES LTD.