

CovaWound™ Hydrocolloid

Hydrocolloid Dressing



✓ **BREATHABLE POLYURETHANE FILM BACKING**

Provides a high rate of moisture vapour transmission.
Acts as a barrier to external contamination.

✓ **ABSORBENT HYDROCOLLOID MATRIX**

Absorbs exudate to form a cohesive gel to create a moist wound environment.

WHAT IS IT?

CovaWound™ Hydrocolloid and CovaWound™ Hydrocolloid (Thin/Transparent) dressings have a two layer construction composed of an absorbent hydrocolloid matrix laminated to a breathable polyurethane film backing. The hydrocolloid matrix absorbs exudate to form a cohesive gel - while the breathable polyurethane film backing provides a high rate of moisture vapour transmission. The combination of these two features ensures that CovaWound™ Hydrocolloid provides an optimal moist wound environment conducive to wound healing and minimizes the chance of damage to healthy peri-wound skin.

WHEN DO I USE IT?

CovaWound™ Hydrocolloid dressing is indicated for the management of lightly exuding wounds such as; pressure, leg and foot ulcers, superficial partial-thickness burns, donor sites, diabetic ulcers, post-operative wounds and skin abrasions. CovaWound™ Hydrocolloid dressing may also be used on small wounds towards the end of the healing phase.



CovaWound™
Advanced Wound Care

1-877-711-6055 | covawound@covalon.com

covalon
TECHNOLOGIES LTD.

HOW TO APPLY:

1. Clean the wound area with normal saline or distilled water.
2. Gently dry the surrounding skin and ensure that it is free of any residue.
3. Choose the size and shape of the dressing which is most suitable, the dressing should extend 2 cm beyond the wound margin.
4. Remove the large piece of the release liner from the dressing.
5. Center and apply the exposed part of the dressing over the wound bed.
6. Avoid touching the adhesive surface to minimize risk of contamination.
7. Remove the small piece of the release liner from the dressing and apply.
8. Finalize application of the dressing, making sure that the edges of the dressing adhere well. This initial adhesion will improve as the dressing warms up. If necessary, use medical tape around the edges to avoid roll-up.

FREQUENCY OF CHANGE:

CovaWound™ Hydrocolloid dressing may be left in place for up to 7 days depending on the patient, the condition of the wound and surrounding skin, or as indicated by accepted clinical practice. The accumulative use of CovaWound™ Hydrocolloid dressing should not exceed 30 days.

NOTE:

- As with any medical device, always refer to the full instructions for use prior to application.
- HCPCS codes contained in this document are provided for informational purposes only and represents no statement, promise, or guarantee by Covalon Technologies Ltd. concerning levels of reimbursement, payment, or charge.

AVAILABILITY:

COVAWOUND™ HYDROCOLLOID

Hydrocolloid Dressing

Exudate Level Key: Light



Reference Number	Size (cm)	Dressings per Carton/ Cartons per Case	HCPCS
5000505	5cm x 5cm	10/20	A6234
5000510	5cm x 10cm	10/20	A6234
5000520	5cm x 20cm	10/20	A6234
5001010	10cm x 10cm	10/20	A6234
5001515	15cm x 15cm	10/20	A6235
5002020	20cm x 20cm	10/20	A6236

Hydrocolloid Dressing (Thin/Transparent)

Exudate Level Key: Light



5200505	5cm x 5cm	10/20
5200510	5cm x 10cm	10/20
5200520	5cm x 20cm	10/20
5201010	10cm x 10cm	10/20
5201015	10cm x 15cm	10/20
5201025	10cm x 25cm	10/20
5201515	15cm x 15cm	10/20
5201518	15cm x 18cm	10/20
5201520	15cm x 20cm	10/20

Suggested Wound Tissue Type:

Necrotic (After Debridement)	✓
Sloughy	✓
Granulating	✓
Epithelialising	✓
Infected	—

Availability may vary by jurisdiction.



CovaWound™
Advanced Wound Care

1-877-711-6055 | covawound@covalon.com

covalon
TECHNOLOGIES LTD.