


CovaWound™ Silicone

Self-Adherent Soft Silicone Foam Dressing

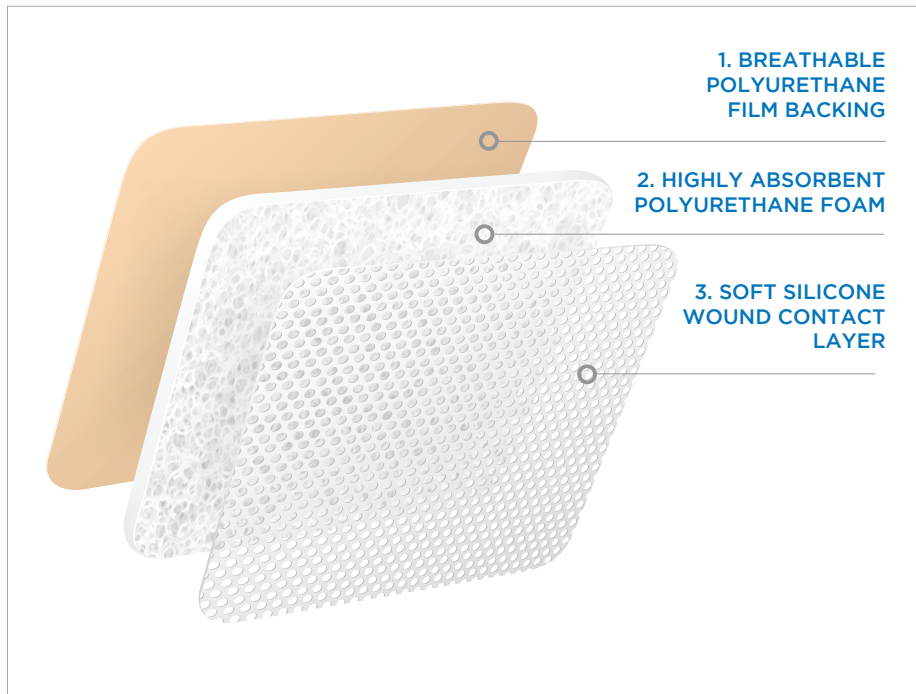

A self-adherent soft silicone foam dressing that absorbs exudate, maintains a moist wound healing environment and prevents epidermal stripping or pain upon removal. Highly conformable and easy to apply, the silicone surface allows the dressing to be easily applied and positioned on the wound.

A three-layer atraumatic dressing

1. BREATHABLE POLYURETHANE FILM BACKING

Allows moisture vapour transmission away from the dressing. Designed to protect against external contamination.

2. HIGHLY ABSORBENT POLYURETHANE FOAM

Designed to absorb and retain exudate, while still maintaining a moist wound environment.

3. SOFT SILICONE WOUND CONTACT LAYER

Facilitates application and retention of the dressing to intact skin, but will not stick to the surface of a wound or cause trauma to delicate new tissue upon removal.

Exudate level

🔴 🔴 🔴 Heavy

Exudate level indicates the highest level of exudate that the dressing can accommodate.

Product Codes and Sizes: Available to order through NHS Supply Chain Advanced Wound Care Framework and Alliance Healthcare

Dressing Size (cm)	Pack Size	Covalon Article Number	PIP Code	Catalogue Code
5 x 5	10	1000505	406-0935	ELA951
5 x 7	10	1000507		ELA952
10 x 10	10	1001010	406-0927	ELA941
12.5 x 12.5	10	1001212		ELA945
10 x 20	10	1001020	406-0893	ELA944
15 x 15	10	1001515	406-0919	ELA947
15 x 20	10	1001520		ELA948
20 x 20	10	1002020	406-0901	ELA949
20 x 50	10	1002050		ELA950

Indications for Use

CovaWound™ Silicone dressing is indicated for the management of moderate to heavily exuding wounds, such as leg and foot ulcers, pressure ulcers, traumatic and surgical wounds, and superficial and partial thickness burns.

Application

- Cleanse the wound in accordance with normal procedures.
- Dry the surrounding skin thoroughly.
- Remove the release liner and apply the adherent side to the wound. Do not stretch.
- For best results CovaWound™ Self-Adherent Soft Silicone Foam Dressing's wound pad should overlap the wound edges by at least 2 cm.

CovaWound™ Advanced Wound Care Line



CovaWound™
Silicone & Silicone Lite



CovaWound™
Silicone & Silicone Lite




CovaWound™
Alginate



CovaWound™
Foam



CovaWound™
Foam

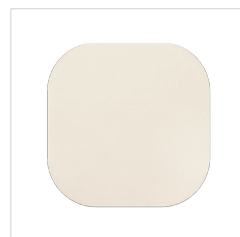

CovaWound™
Hydrocolloid &
Hydrocolloid Thin /
Transparent



CovaWound™
SuperAbsorbent



MediClear™
Scar

Covalon Technologies (Europe) Ltd
Lion Court, Staunton Harold Hall,
Melbourne Road, Ashby-de-la-Zouch LE65 1RT
+44 (0)1332 695020 www.covalon.co.uk

CW100-UK-3 (4/2018)

covalon
TECHNOLOGIES LTD.

CovaWound is a trademark of Covalon Technologies Ltd.