

CovaWound™ SuperAbsorbent

Capillary Action Dressing



✓ **FLUID-REPELLENT NONWOVEN BACKING**

Acts as a barrier and prevents exudate strike-through.

✓ **SUPERABSORBENT POLYMER (SAP) CORE**

Locks in fluid even under pressure, decreasing the risk of maceration of the wound and surrounding tissue.

✓ **SOFT HYDROPHILIC WOUND CONTACT LAYER**

Enables transmission of fluid through to the SAP core.

WHAT IS IT?

CovaWound™ SuperAbsorbent dressing consists of a soft polyethylene (PE) wound contact layer, a Polyurethane (PU) foam with SuperAbsorbent Polymer (SAP, polyacrylate) which provides excellent fluid absorption, and a non-woven fiber top layer. The non-adherent hydrophilic wound contact layer enables transmission of fluid through to the SAP core, which locks in fluid, decreasing the risk of maceration of the wound and surrounding tissue.

WHEN DO I USE IT?

CovaWound™ SuperAbsorbent dressing is indicated for the management of moderate to heavily exuding wounds such as leg ulcers, foot ulcers, pressure ulcers, traumatic and surgical wounds, and first and second degree burns.



CovaWound™
Advanced Wound Care

1-877-711-6055 | covawound@covalon.com

covalon
TECHNOLOGIES LTD.

CovaWound™ SuperAbsorbent

HOW TO APPLY:

1. Cleanse the wound in accordance with normal procedures.
2. Dry the surrounding skin thoroughly.
3. Choose the size and shape of the dressing which is most suitable, the dressing should extend approximately 2 cm beyond the wound margin.
4. Remove the dressing from the packaging and apply with the white non-adherent hydrophilic layer to the wound.
5. Secure the dressing with a suitable bandage or tape.

FREQUENCY OF CHANGE:

CovaWound™ SuperAbsorbent dressing may be left in place for up to 7 days depending on the patient, the condition of the wound and surrounding skin, or as indicated by accepted clinical practice. The accumulative use of CovaWound™ SuperAbsorbent dressings should not exceed 30 days.

NOTE:

- As with any medical device, always refer to the full instructions for use prior to application.
- HCPCS codes contained in this document are provided for informational purposes only and represents no statement, promise, or guarantee by Covalon Technologies Ltd. concerning levels of reimbursement, payment, or charge.

AVAILABILITY:

COVAWOUND™ SUPERABSORBENT

Capillary Action Dressing

Exudate Level Key: Heavy



Reference Number	Size (cm)	Dressings per Carton/ Cartons per Case	HCPCS
4000575	5cm x 7.5cm	10/10	
4007575	7.5cm x 7.5cm	10/10	
4001010	10cm x 10cm	10/10	A6251
4001220	12cm x 20cm	10/10	A6252
4001515	15cm x 15cm	10/10	
4002020	20cm x 20cm	10/10	A6252
4002030	20cm x 30cm	10/10	A6253

Suggested Wound Tissue Type:

Necrotic (After Debridement)	✓
Sloughy	✓
Granulating	✓
Epithelialising	✓
Infected	—

Availability may vary by jurisdiction.



CovaWound™
Advanced Wound Care

1-877-711-6055 | covawound@covalon.com

covalon
TECHNOLOGIES LTD.