

# MediClear™ Scar

Self-Adherent Silicone Film Dressing for Scar Care

- ✓ Intimately adheres to the contours of the scar to help prevent and/or manage hypertrophic and keloid scars.
- ✓ Can be worn comfortably and continuously for up to 7 days.
- ✓ No additional fixation required.
- ✓ Thin, discreet and flexible.



#### POLYURETHANE FILM BACKING

- Thin and discreet.
- Conformable.
- Breathable with good moisture vapour exchange.
- Effective barrier against external contamination.
- Waterproof.

#### SOFT SILICONE CONTACT LAYER

- Adheres gently but securely to scar tissue and skin.
- Ensures removal with minimal pain and trauma.
- No extra fixation required.

#### WHAT IS IT?

MediClear™ Scar is a thin, discreet, clear medical self-adherent silicone film dressing consisting of a polyurethane film coated with a soft silicone adhesive gel that intimately adheres to the contours of the scar to help prevent and/or manage hypertrophic and keloid scars.

#### HOW DOES IT WORK?

MediClear™ Scar is designed as a prophylactic therapy on closed wounds to help prevent hypertrophic or keloid scarring as well as for the management of both old and new hypertrophic and keloid scars, including burn and surgical scars.



## MediClear™ Scar

Self-Adherent Silicone Film Dressing for Scar Care

1-877-711-6055 | [info@covalon.com](mailto:info@covalon.com)

**covalon**  
TECHNOLOGIES LTD.

# MediClear™ Scar

## AREAS OF USE:

On surgical incisions, use MediClear™ Scar only after sutures have been removed and the site is fully closed. MediClear™ Scar is for external use only. Do not use ointments or creams under the dressing as they may interfere with the performance of the product. The skin should be dry and free of detergent residue to prevent skin irritation and to ensure good adhesion. Do not use on open wounds or where the skin integrity is compromised.

## HOW TO APPLY:

1. Ensure the scar and surrounding skin is clean and dry before MediClear™ Scar application.
2. Open the packaging and remove the dressing.
3. Peel one side of the white liner, exposing the silicone adhesive. If needed use the frame's tab to avoid touching the adhesive.
4. Place the exposed adhesive on the skin and smooth the dressing over the skin. Avoid stretching when applying over joints.
5. Hold the flap of the remaining white liner and peel off and smooth the dressing over the skin towards the edges. Apply continuous light pressure until the entire dressing is adhered to the skin.
6. To remove frame; start from the pre-cut line, peel off in a circular motion and smooth the dressing towards the edges to ensure good adhesion.

## FREQUENCY OF CHANGE:

MediClear™ Scar should optimally be worn for 24 hours a day for up to 7 days, and should be changed as necessary or if the dressing becomes loose, soiled, or compromised in any way. Total therapy time should be in the range of 2 to 4 months. MediClear™ Scar is waterproof.

## NOTE:

As with any medical device, always refer to the full instructions for use prior to application.

## AVAILABILITY:

### MEDICLEAR™ SCAR

#### Self-Adherent Silicone Film Dressing for Scar Care

Reference Number	Size (cm)	Dressings per Carton Cartons per Case
9000404	4cm x 4cm	5/10
9000607	6cm x 7cm	5/10
9000620	6cm x 20cm	5/10
9000625	6cm x 25cm	5/10
9000630	6cm x 30cm	5/10
9001010	10cm x 10cm	5/10
Nipple 9000285	Ø 8.5cm	2 dressings per pouch / 5 pouches per carton 10 cartons per case
Keyhole Shape 9008585	Ø 8.5cm + 3.25cm x 8.5cm	2 dressings per pouch / 5 pouches per carton 10 cartons per case



## MediClear™ Scar

Self-Adherent Silicone Film Dressing for Scar Care

1-877-711-6055 | info@covalon.com

**covalon**  
TECHNOLOGIES LTD.