

innovation in scope

OTSC[®] System.... saving lives[®]

The innovative clipping system for flexible endoscopy



The OTSC[®] System stands for superior clinical efficacy¹⁻⁶, easy and quick application⁶ and cost effectiveness^{3, 7}.

The OTSC[®] System is used in flexible endoscopy for

- acute bleeding
- wall closure
- closure of chronic lesions
- management of complications after endoscopic or surgical procedures

OTSC® System

.

The OTSC® System is an innovative clipping system to be applied The exceptional features and therapeutic functions of the via flexible endoscopes. It offers the physician unique features superior to any other device:

- dynamic compression, continuous adaption to tissue thickness
- larger volume of tissue secured
- higher stability at the lesion site
- minimal strain on surrounding tissue

..... Application

Hemostasis



- 1 Jensen DM, Kovacs T, Ghassemi KA, Kaneshiro M, Gornbein J. Randomized Controlled Trial of Over-the-Scope Clip as Initial Treatment of Severe Nonvariceal Upper Gastrointestinal Bleeding. Clin Gastroenterol Hepatol. 2021 Nov;19(11):2315-2323.e2
- 2 Meier B, Wannhoff A, Denzer U, Stathopoulos P, Schumacher B, Albers D, Hoffmeister A, Feisthammel J, Walter B, Meining A, Wedi E, Zachäus M, Pickartz T, Küllmer A, Schmidt A, Caca K. Over-the-scope-clips versus standard treatment in high-risk patients with acute non-variceal upper gastrointestinal bleeding: a randomised controlled trial (STING-2). Gut. 2022 Jul;71(7):1251-1258

OTSC® System are based on its unique material and design: the superelastic Nitinol[®] is biocompatible, MRI conditional, and, if needed, even suited to be applied as a long-term implant.




The use of the OTSC[®] System is intuitive and easy, very similar to other devices based on the application cap principle. The application cap is mounted to the tip of the endoscope. By turning the handwheel, the thread is tightened and the clip is applied. It is compatible with commercially available endoscopes. A variety of different sizes and combinations are available for commonly used endoscopes.

Our application aids allow easier positioning of lesions: the OTSC[®] Anchor (e.g. fistula or ulcer bleeding treatment) and the OTSC[®] Twin Grasper[®] (e.g. perforation closure) assist you in approximating tissue precisely into the cap.

All $\mathsf{OTSC}^{\circledast}$ products are disposable and designed for single patient use.

Perforation closure

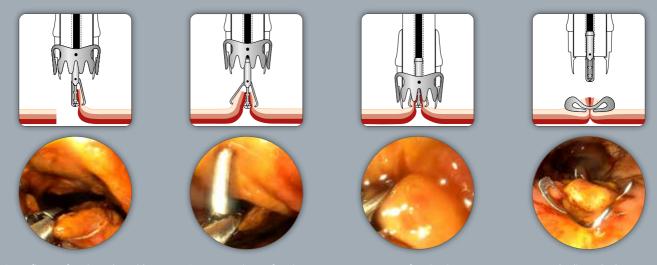

Grasp first perforation edge with one of the two OTSC® Twin Grasper® jaw parts.

Grasp opposite perforation edge with second jaw part.

Retract perforation into cap OTSC® Twin Grasper® must b fully inside cap).


Apply clip and release OTSC[®] Twin Grasper[®].

Closure of a perforation in the rectum, source: Dr. Mauro Manno & Dr. Paola Soriani, UOSD Endoscopia Digestiva Area Nord, Azienda USC di Modena, Ospedale di Carpi e Mirandola, Italy

- 3 Lau J, Tan CH, Sun X, Song H, Li L, Li R, Li P, Feng J, Wang B, Leung W.K, Hartley I, Moss AC, Suen BY, Yu Y, Chan FK. Over-the-scope clips (OTSC) versus standard endoscopic treatment in patients with acute non-variceal upper gastrointestinal bleeding. UEG Week Virtual (October 3-5) 2021
- 4 Wedi E, Fischer A, Hochberger J, Jung C, Orkut S, Richter-Schrag HJ. Multicenter evaluation of first-line endoscopic treatment with the OTSC in acute non-variceal upper gastrointestinal bleeding and comparison with the Rock all cohort: the FLETRock study. Surg Endosc. 2018 Jan;32(1):307-314.
 5 Schmidt A, Gölder S, Goetz M, Meining A, Lau J, von Delius S, Escher M, Hoffmann A, Wiest R, Messmann H, Kratt T, Walter B, Bettinger D, Caca K. Over the Scope Clips Are
- 5 Schmidt A, Gölder S, Goetz M, Meining A, Lau J, von Delius S, Escher M, Hoffmann A, Wiest R, Messmann H, Kratt T, Walter B, Bettinger D, Caca K. Over the Scope Clips Are More Effective Than Standard Endoscopic Therapy for Patients With Recurrent Bleeding of Peptic Ulcers. Gastroenterology. 2018 Sep;155(3): 674-686.e6.



Fistula closure

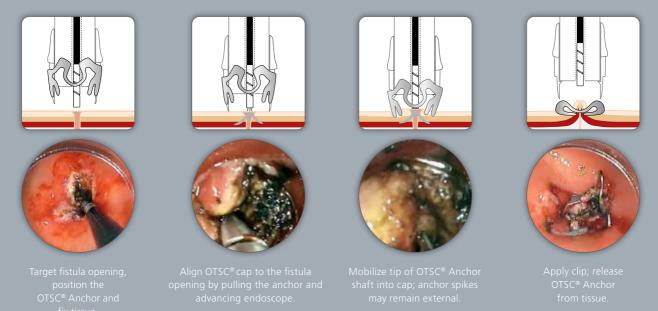


- 6 Bapaye J, Chandan S, Le Naing Y, Shehadah A, Deliwala S, Bhalla V, Chathuranga D, Okolo PI 3rd. A systematic review and meta-analysis of safety and efficacy of over the scope clips versus standard therapy for high-risk non-variceal upper gastrointestinal bleeding.
 7 Küllmer A, Behn J, Glaser N, Thimme R, Caca K, Schmidt A. Over-the-scope clips (OTSC) are cost-effective in recurrent peptic ulcer bleeding. United European Gastroenterol J. 2019 Nov; 7(9): 1226-1233.

one & done

OTSC[®] – superior outcome in hemostasis

Three randomized controlled trials (USA, Europe, Asia) on first-line OTSC[®] therapy demonstrate a significant clinical benefit compared to former standard treatment. Clinical success rates:

96.0% vs. 71.4% (p=0.017) in ulcer & Dieulafoy; Jensen DM et al., Clin Gastroenterol Hepatol 2021¹ 91.7% vs. 73.1% (p=0.019) in high-risk lesions; Meier B et al., Gut 2022² 96.8% vs. 86.5% (p=0.007) in all cases of NVUGIB; Lau J et al., UEGW 2021³

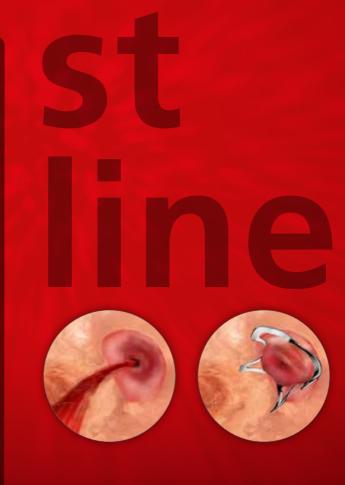





. OTSC[®] System

0 0 0 <mark>0 v e s c 0</mark>

Details and components

The OTSC[®] System consists of an applicator cap with a mounted clip, handwheel and thread retriever. The OTSC[®] System is available in the following variety of cap sizes and clip designs to provide secure application regardless of the anatomical situation and endoscope type.

- 4 different cap and corresponding clip sizes
- 2 different cap depths for capturing more or less tissue
- 3 different teeth shapes suitable for different areas of application

OTSC® CLIP	M	a stalle	AND	162)
OTSC [®] Version	mini	11	12	14
Endoscope diameter Ø [mm]	8.5-10	8.5 - 11	10.5 – 12	11.5 – 14
Depth of cap 3 mm		11/3a ¹ (100.03)*	12/3a ¹ (100.05)*	14/3a ² (100.07)*
		11/3t ¹ (100.04)*	12/3a ² (100.28)*	14/3t ² (100.08)*
			12/3t ¹ (100.06)*	
			12/3t ² (100.29)*	
6 mm	mini/6a¹ (100.01)*	11/6a ¹ (100.09)*	12/6a ¹ (100.11)*	14/6a ² (100.13)*
	mini/6t ¹ (100.02)*	11/6t ¹ (100.10)*	12/6a ² (100.30)*	14/6t ² (100.14)*
			12/6t1(100.12)*	
			12/6t ² (100.31)*	
			12/6gc1(100.27)*	
Max. outer diameter Ø [mm]	14.6	16.5	17.5	21
		* (ref. no	.) ¹ thread length 165 cn	n ² thread length 220 cm
type a	type t		type gc	
blunt teeth, primarily compression effect	teeth with small spikes, compression and anchoring effect		elongated teeth w application: closur	

Application aid

There are two instruments available to enable a more effective application. These can be inserted next to the thread in the same working channel.



For better approximation of tissue especially when hard or fibrotic (e.g. fistulae, chronic ulcer), OTSC[®] Anchor 220tt especially for thin tissue.

	OTSC [®] Anchor	OTSC [®] Anchor 220tt
ref. no.	200.10	200.11
working length	1650 mm	2200 mm
max. Ø	2.4 mm	2.4 mm
needle width	12 mm	9 mm
stitch depth	4 mm	2-2.5 mm



For easier approximation of gaping edges of a lesion (e.g. perforation).

	OTSC [®] Twin Grasper [®]		
ref. no.	200.44	200.45	
working length	1650 mm	2200 mm	
max. Ø	2.6 mm	2.6 mm	

Ovesco Endoscopy AG Friedrich-Miescher-Str. 9

72076 Tuebingen Germany

 Phone
 + 49(0)7071 96528-160

 Fax
 + 49(0)7071 96528-260

 E-Mail
 service@ovesco.com

Ovesco Endoscopy USA Inc.

15300 Weston Parkway Suite 101 Cary, NC 27513, USA

Phone +1 919 651 9449 Fax +1 408 608 2077 customerservice@ovesco-usa.com

Scan here for more product information.



www.ovesco.com

CE⁰¹²⁴ English version 12 | 2022-09-28

Ovesco Endoscopy AG is a medical device company specialising in the fields of flexible endoscopy, endoluminal surgery and telemetric diagnostics. The company develops, manufactures and markets innovative products for the detection and treatment of gastrointestinal disease. Ovesco products are synonymous with efficacy and efficient application.