MOSKINTO®

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EN NOTES ON USE

Dear customer, thank you for choosing MOSKINTO. Should you have any questions, please contact our Customer Service via our website: www.moskinto.com

KEY TO THE ICONS

CE The design of the product corresponds to that of a medical device of class 1.

- Latex-free
- Do not use the plaster twice, Adhesive strength decreases.
- Read the instruction manual on Use carefully.
- Protect the plaster strips from direct sunlight.
- Store the plaster strips in a dry and clean place.

MD Medical Device

Dispose professionally

USE

The plaster strips are used to relief itching and swelling after mosquito bites. Stick the Moskinto patch directly over your insect bite, do not scratch before applying. The itching should start to subside in a short time after application. The plaster comes off automatically after 4 to 7 days.

CAUTION!

- Do not stick the plaster on open wounds.
- In case of allergic reactions remove the patch and consult a doctor.

CHARACTERISTICS OF THE PLASTER STRIPS

The plaster strips consist of polyester, polyurethane and acrylic adhesive. Moskinto is water-resistant and can be worn for several days and remains in place when taking showers or baths. Moskinto does not contain any medications or active ingredients. **Without chemical ingredients.**

DISPOSAL

The packaging material is reusable. Please dispose of the packaging in environmentally friendly manner and return it to the recycling collection. Please also dispose of the articles in an environmentally friendly manner. Contact your local municipal administration for more information.

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